



The Midmark Fiber-optic  
Highspeed Swivel Coupler



Manufactured and/or  
Distributed by Midmark Corporation  
60 Vista Drive  
Versailles, Ohio 45380  
1-800-MIDMARK  
[midmark.com](http://midmark.com)

003-2910-00 Rev. AA1 7/20  
70379




## **O-RING REPLACEMENT INSTRUCTIONS**

O-Ring Kit  
Item: 016-1731-00

# O-RING REPLACEMENT INSTRUCTIONS



- Replace ALL O-rings, every time.
- Remove O-rings in order shown using explorer.
- Discard used O-rings.
- Remove any debris with a clean gauze and isopropyl alcohol only.
- Install new O-rings in reverse order shown.
- Spray Midmark Conditioner onto all O-rings. Ensure even coating of all O-rings. Remove excess conditioner with a towel or gauze.

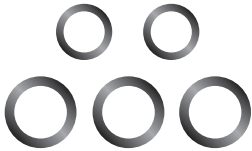

O-rings