

# Reed Valve Replacement

## Midmark Models: P52 and P72

[77001450]

*Applies to Models:*  
P52 (all)  
P72 (all)



### WARNING


*Disconnect pump at electrical power. Vent air lines to the compressor to remove pressure.*

**Step 1:** Disconnect main power supply.


**Step 2:** Remove old reed valve plates and gaskets from head.

### Installation

**Step 4:** Torque head bolts to 53 in-lbs.

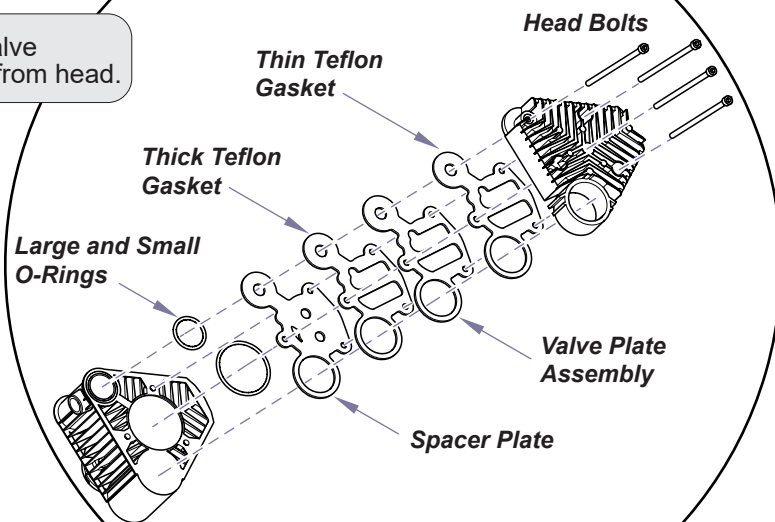

**Installation**  
**Step 3:** Install new reed valve plates and gaskets in order shown.

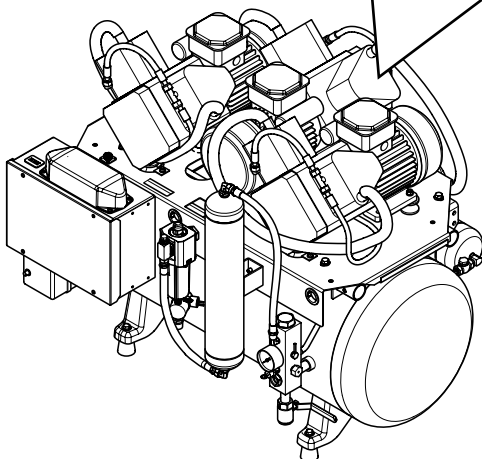

### Installation

**Step 5:** Connect main power supply.