

## Occlusion Valve with Safety Relief [93805114]

**Components:**  
Occlusion Valve..... 1



### Caution

Do not activate Oxygen flush when valve is occluded.

Safety relief pressure - 25-30 cm - H<sub>2</sub>O.

Replacement Diaphragm and Backup Ring Kit available, P/N 91805115.

### To attach occlusion valve:

- Remove existing APL valve from Dome.
- Thread APL valve into top of Occlusion Valve.
- Thread Occlusion Valve into top of Dome.
- Tighten parts hand-tight only.



### WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

