

Manifold Check Valve Replacement [10563600]




WARNING

Open drain valve to completely depressurize compressor tank prior to removing manifold cap and check valve.

Removal

Step 1: Disconnect power.
Open drain valve.

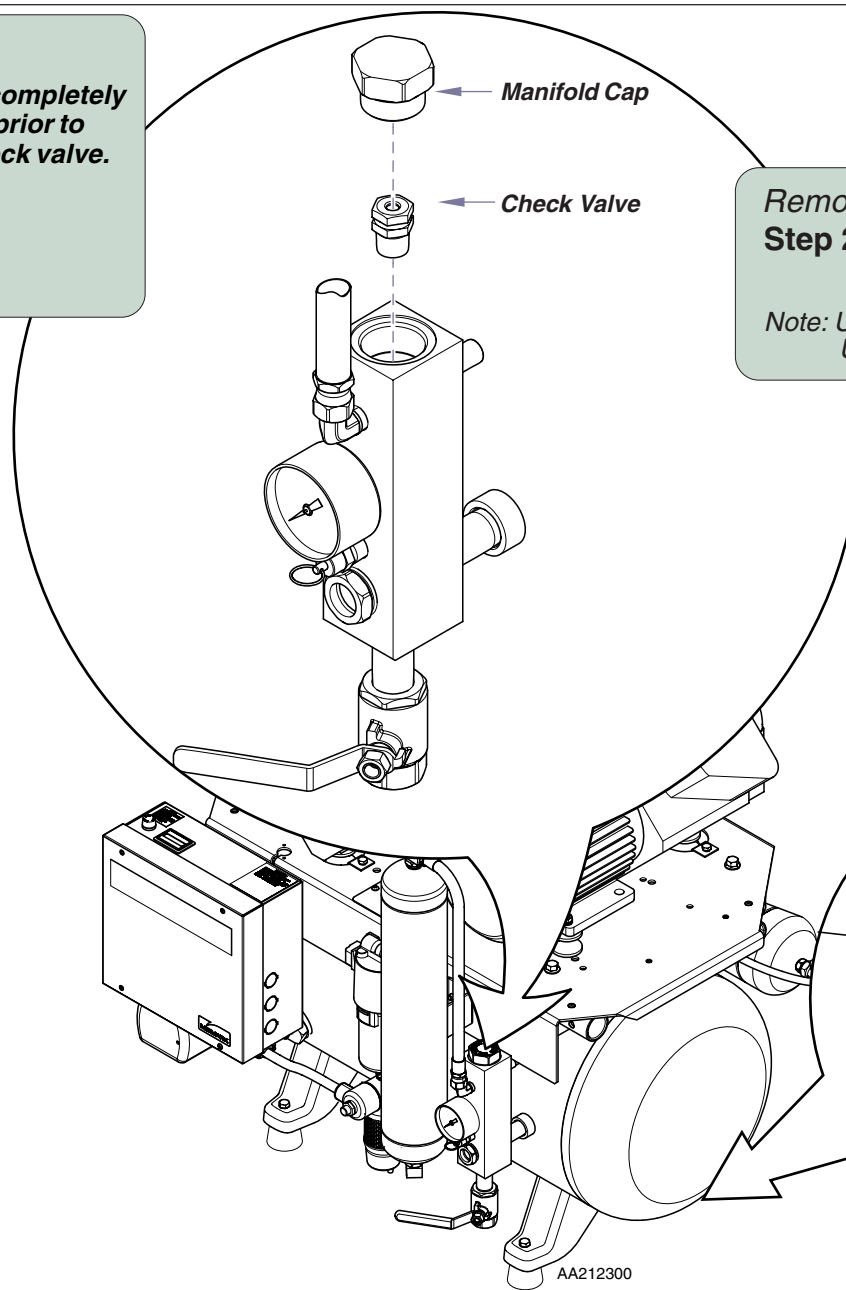

Removal

Step 2: Remove manifold cap.
Remove check valve.

*Note: Use Ratchet with extension and 3/4" socket to loosen valve.
Use needle nose pliers to lift check valve out of block.*

Installation

Step 3: Apply thread sealant to check valve.
Install check valve.
Replace manifold cap.

Drain Valve

Installation

Step 4: Close drain valve.
Connect power.