



# Bi-Annually

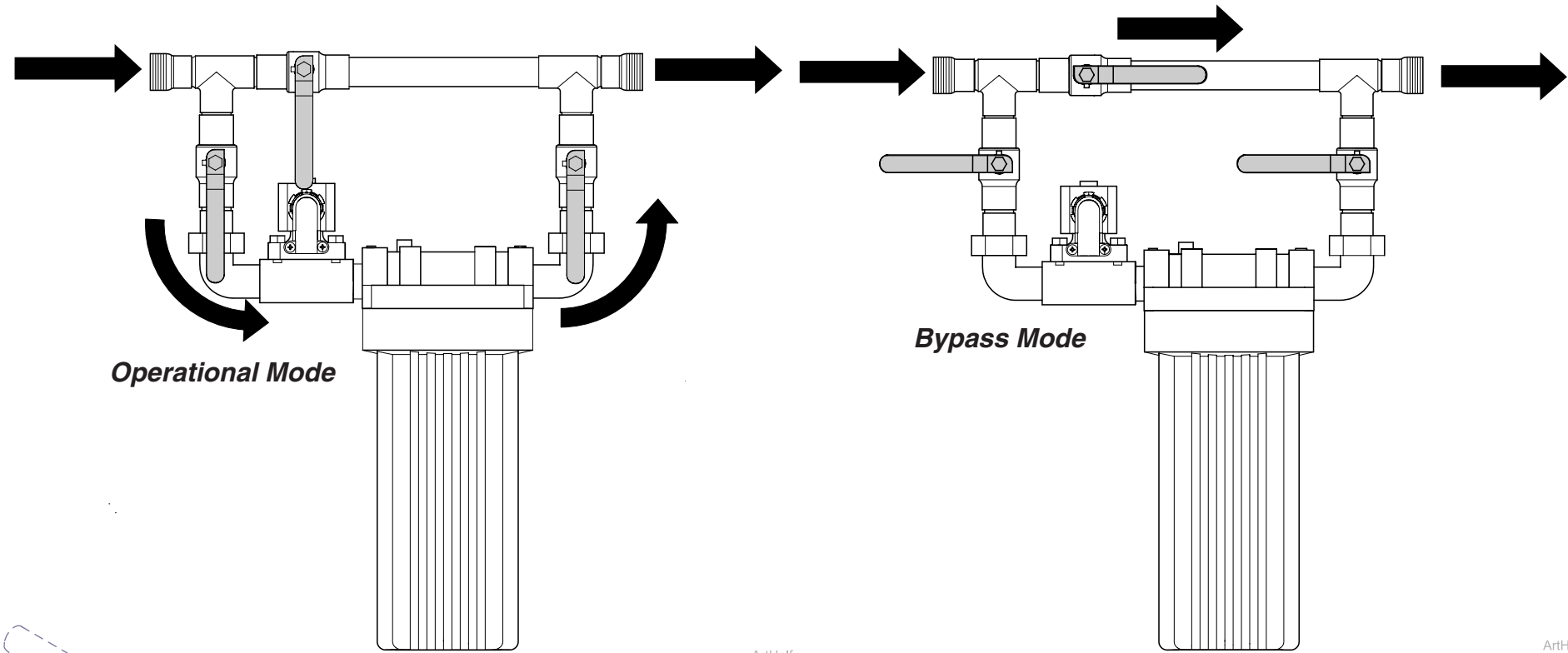



## Caution

Solenoid may be hot. Use care when adjusting shut-off valves and/or performing maintenance. Failure to comply may result in personal injury due to burns.

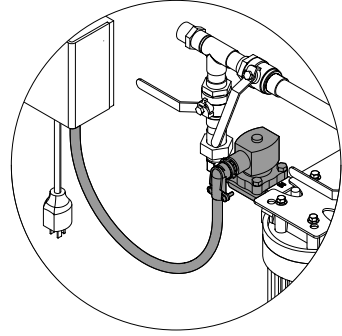

## A. Replace Filter.

- Set valves in Bypass Mode.
- Depressurize system by pushing button on cover.
- Remove bowl using wrench.
- Replace filter.
- (Verify O-ring is seated inside bowl before reinstalling.)
- Assemble bowl. Tighten with wrench. (Do not overtighten)
- Set valves in Operational Mode.
- Check system for leaks.

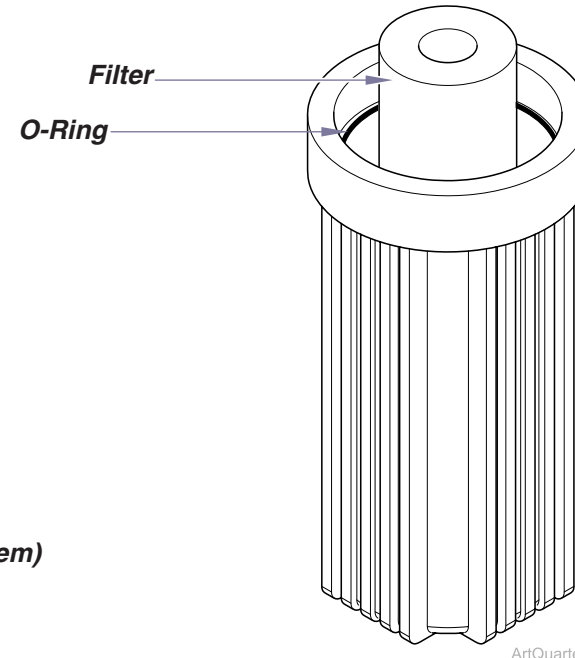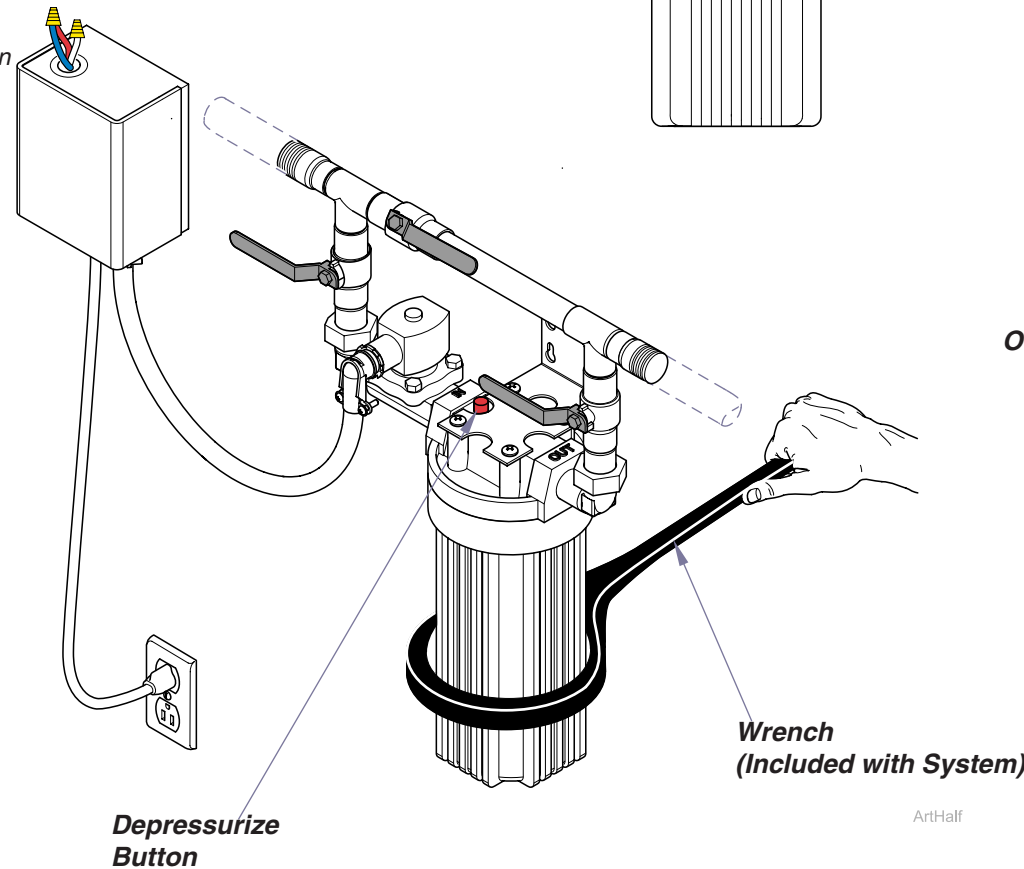

## Midmark Corporation

60 Vista Drive  
Versailles, Ohio 45380 USA

1-800-643-6275

1-937-526-3662

[midmark.com](http://midmark.com)

