


Oil-Less Compressor Maintenance Kit

Models P52, P72
[77001610, 77001611]

Components:

O-Ring, Spare O-Ring,
Element Filter
Intake Filters (2 per head)

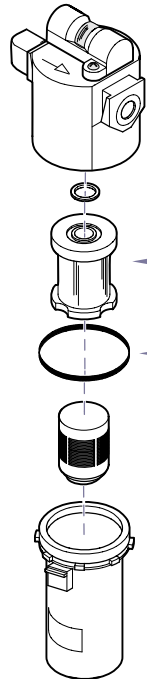

O-ring and Filter

Step 1: Disconnect power to compressor.
Drain storage tank of all air and liquid.

← **Element Filter**

← **O-Ring**

O-ring and Filter

Step 2: Push down on release button, turn slightly and pull to remove filter bowl.

O-ring and Filter

Step 3: Replace filter and o-ring. Reinstall bowl and lock into place.

Intake Filter

Step 4: Replace both filters on each head.

- Firmly grab black boot, twist, rock and pull outward.
- Remove filter from boot.
- Insert new filter fully into rubber filter boot.
- Press into compressor head.
- Firmly press around boot flange to secure in place.

