

Midmark Stack Rack Installation

[77000804]

Intended Use:
 To save mechanical room space by allowing a dental wet-ring vacuum and dental compressor to be mounted one above the other.

Requirements / Restrictions:
 Do not climb or stand on stack rack, it could tip over.
 The top shelf will only support compressors up to 400 lbs.
 (Midmark Models P21, P22, P32, P52, CL21, CL22 & CL32)
 Shelf-mounted compressors require channel mount kit, 77000808.

Components:

Three-member Frame	2	Shelf	1
Single-member Frame	1	Single-member Frame	1
Tapered Post Adapters	10	Separator Mounting Hardware Kit	1
Posts	4	Drain Tubing	10'
Wedge Lock Connectors	4		

Note
 Use a soft mallet to seat adapters and wedge locks into frame and shelf.

Step 5: Attach post adapters to notch 50. Slide three member frame onto post.

Step 6: Attach post adapters to notch 61. Slide single member frame onto post.

Step 4: Seat shelf onto posts.

Step 3: Attach wedge lock connectors to each post. Align pointer with notch on chart that matches compressor model that will be located below shelf.

Step 2: Slide posts through three member frame.

Model	Height
P72	45"
P52	40"
P32	
P22	
P21	
CL52	36"
CL32	
CL22	
CL21	

Step 1: Attach post adapters to posts at notch 13.

Step 7: Adjust feet accordingly.

Step 8: Connect drain tube to barbed fitting under shelf.

