



JB-70 PREVA

Midmark Corporation  
1001 Asbury Drive  
Buffalo Grove, IL 60089  
USA

1-800-MIDMARK  
midmark.com

#### Technical Specifications:



##### Electrical:

JB-70 Line Voltage:

120 VAC 10 % 50/60 Hz Max Line Current 10 A

PREVA Line Voltage:


100-230 VAC 10% 50/60 Hz Max Line Current 10 A)

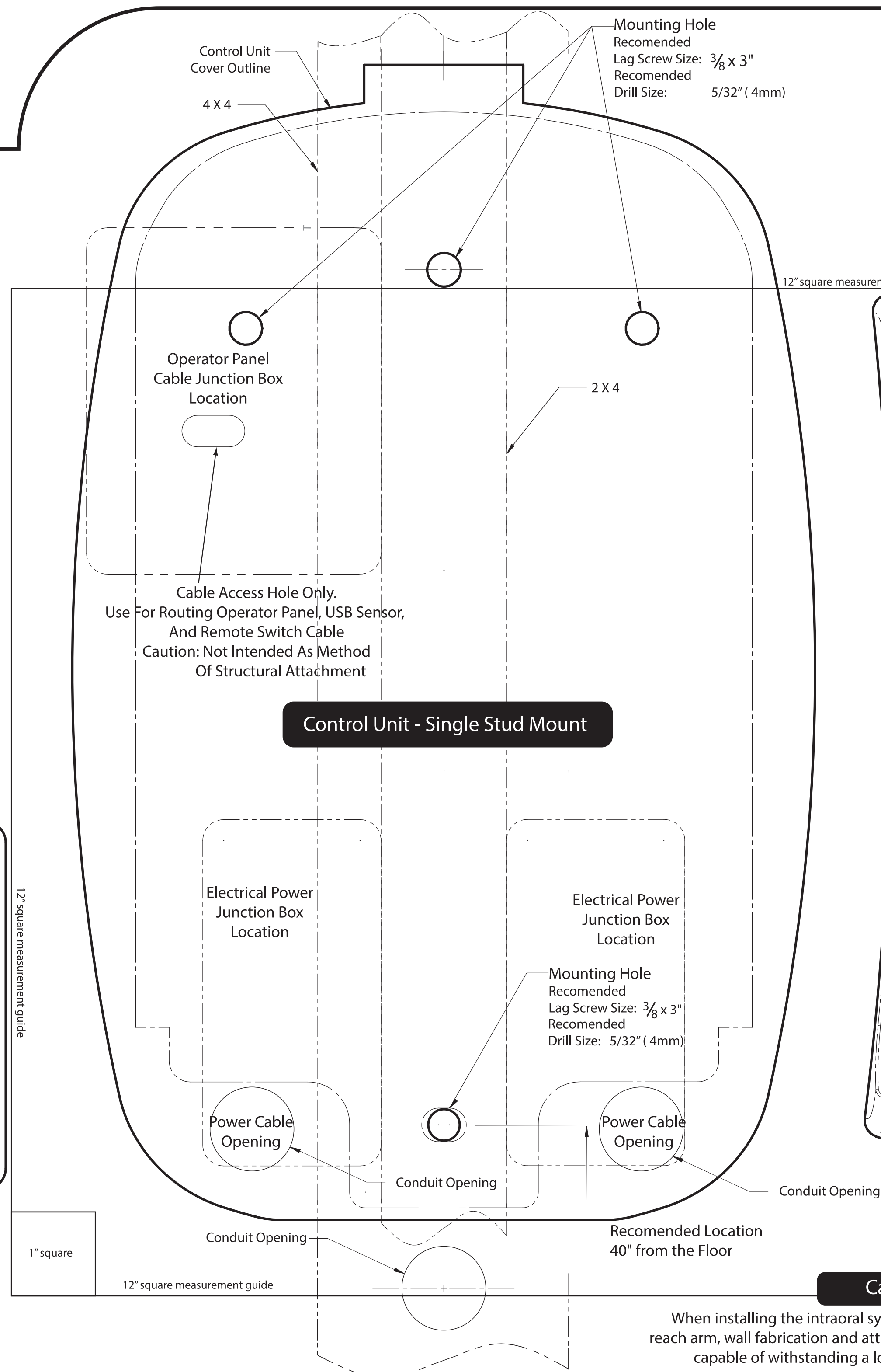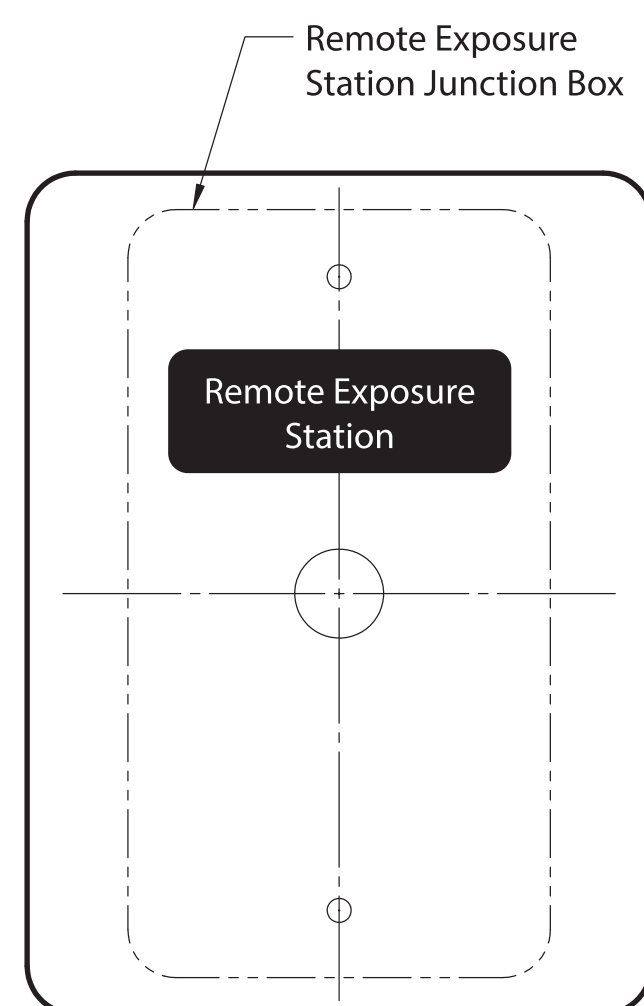

#### Attachment Cautions

When using lag screws as the method of attachment, it is imperative to consider the full scope of the task. Several factors must be considered for safe, permanent installations. For detailed issues see installation manual. Some key issues are:

1. The size of the pilot hole required for the lag screw will be different based on the grade, age and condition of the lumber.
2. Plywood, particle board or similar construction materials, should not be used for attachment.
3. Never over-tighten the lag screws as this will weaken the mechanical connection.
4. Progeny mechanical designs will exert up to 920 ft lbs (127 Kg m) of loading moment on the supporting structure.
5. Inspect the attachment method 30 days after the installation and, every 6 months there after.

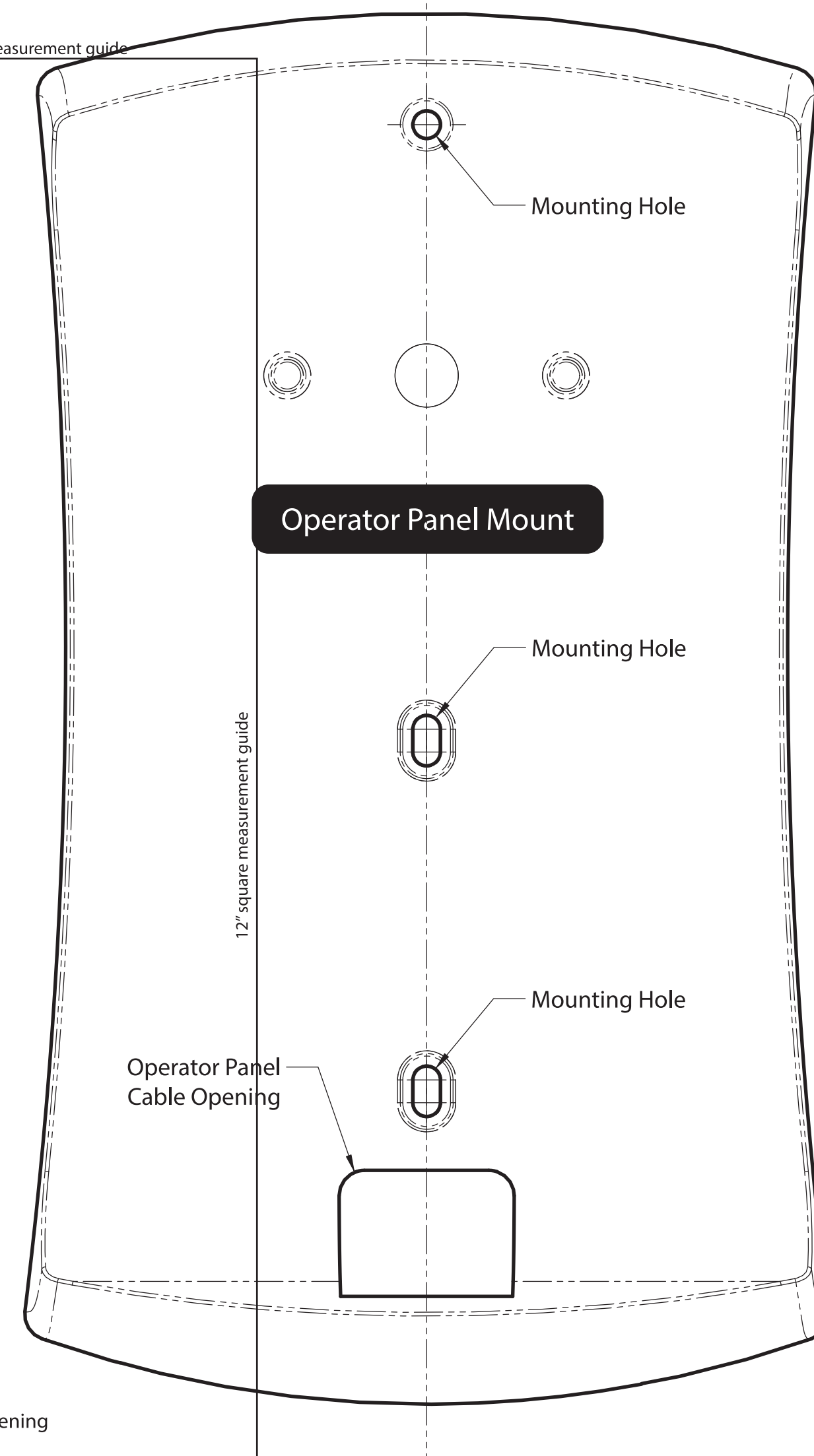

#### Caution:

When installing the intraoral system, combined with the 82 in (208 cm) reach arm, wall fabrication and attachments to the building structure must be capable of withstanding a load moment of 920 ft lbs ( 127 Kg m)

30-S0003  
Rev. S01

Made in USA



JB-70  
PREVA

Catalog Number (Beginning with)	A	B	C	D
DCD5 DCV5 I7015 P7015	24 5/8" (63 cm)	20 1/2" (52 cm)	52 1/2" (133 cm)	56 1/2" (144 cm)
DCD6 DCV6 I7016 P7016	34 5/8" (88 cm)	30 1/2" (78 cm)	62 1/2" (159 cm)	66 1/2" (167 cm)
DCD7 DCV7 I7017 P7017	44 5/8" (114 cm)	40 1/2" (103 cm)	72 1/2" (184 cm)	76 1/2" (194 cm)
DCD8 DCV8 P7018	48 5/8" (124 cm)	44 1/2" (113 cm)	76 1/2" (194 cm)	80 1/2" (204 cm)

