

Control Panel Installation

Contents:

Panel Assembly..... 1
Wall Anchors..... 4
Screws #6 x 1"..... 4
Screws #6-32 x 1/2" 4

Language of origin: English

Intended Use:

To provide a remote means of controlling dental equipment from inside the office area.


Restrictions:

Install control panel according to local codes. Although control panel can be installed directly into sheet rock, many local codes require the use of a junction box.


Note

Control Panels will mount directly to a 3 or 4 gang junction box. See chart in this document to determine correct junction box size.

Step 1: Shut power supply off at circuit breaker box.

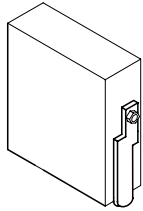


Step 2: Cut out 5" x 2 1/2" area in sheet rock.

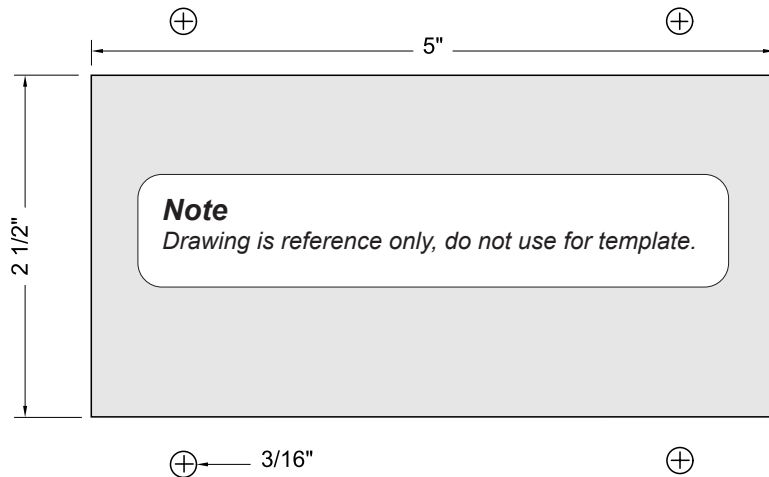

Step 3: Mark mounting hole locations using the front cover as a template. Drill (4) 3/16" holes in spread as shown. Install anchors.


Step 4: Route one 18-3 bell wire from each piece of equipment through opening in sheet rock. Attach control panel wires to color corresponding wires.

Sheet Rock Installation Shown

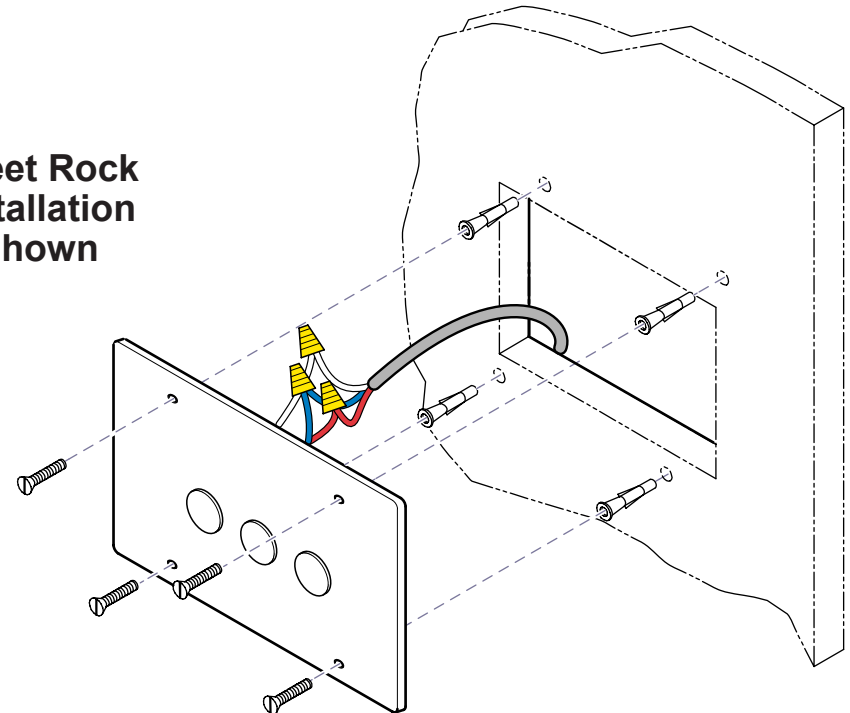


Step 5: Screw front cover to sheet rock.

General Wiring Reference

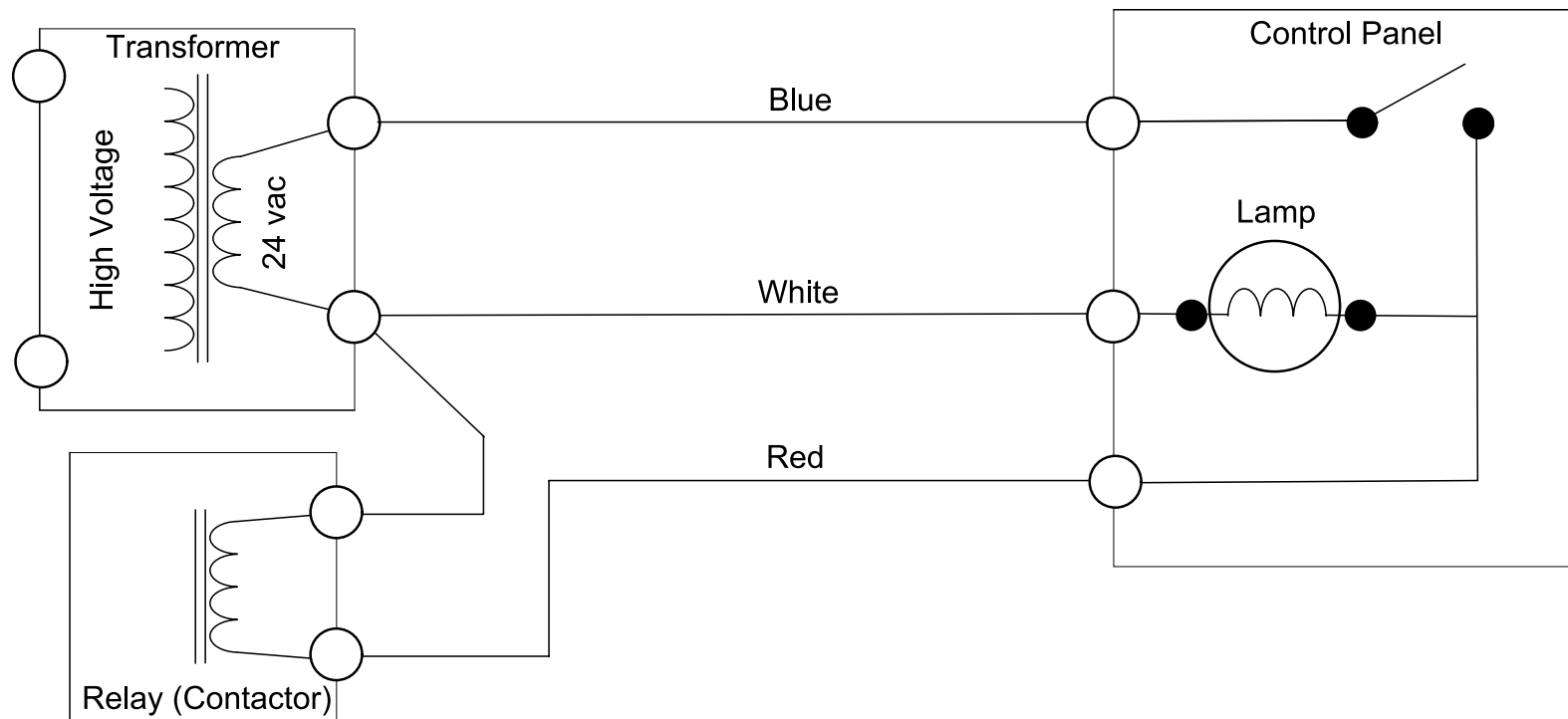


Midmark Wiring Examples


(LED switches shown)

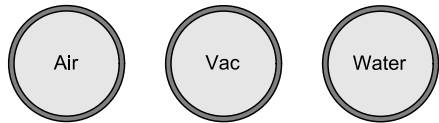

FRONT VIEW

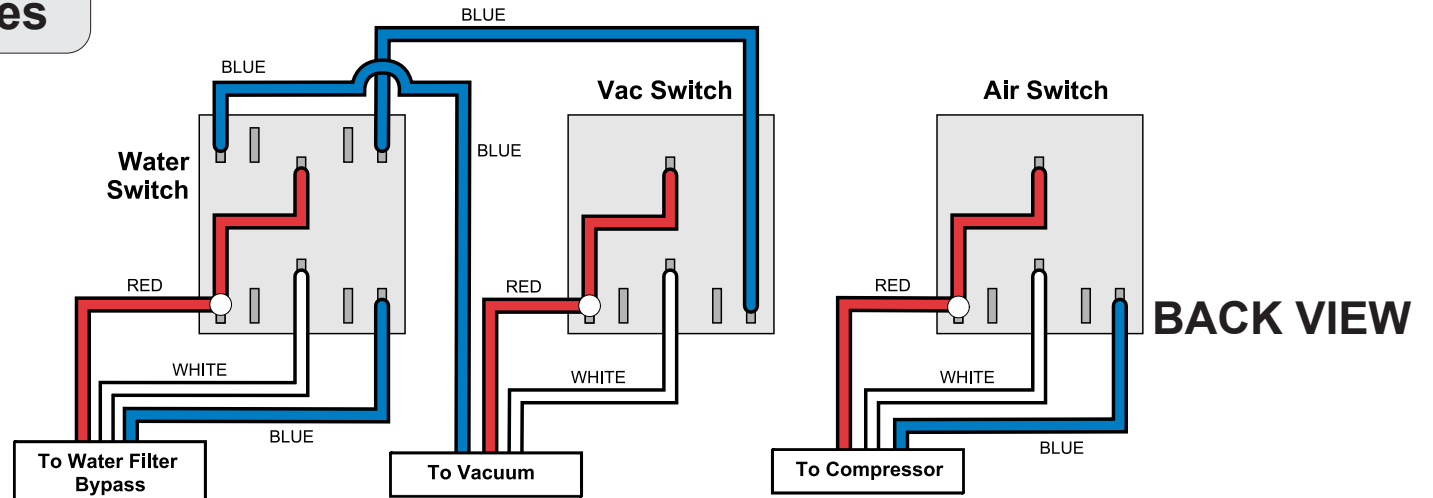

FRONT VIEW

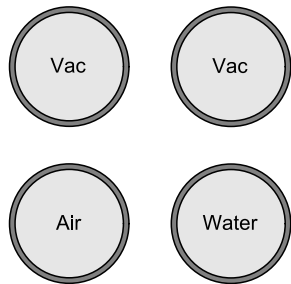

(LED switches shown)

