


WARNING
Please read all mounting instructions carefully! Improper installation may result in injury or damage to the product. Weight limit on the table is 150 lbs.

Hardware not included.

Begin by determining where the table will be installed. The mounting plate of the table is pre-drilled with four holes for bolting the table to the wall. Holes are 16" apart to correspond with standard wall stud placement. If not placing the wall mount table to align with existing wall studs, modifications to the wall will be necessary. Even when mounting onto studs we recommend reinforcing the wall by securing a board between the studs.

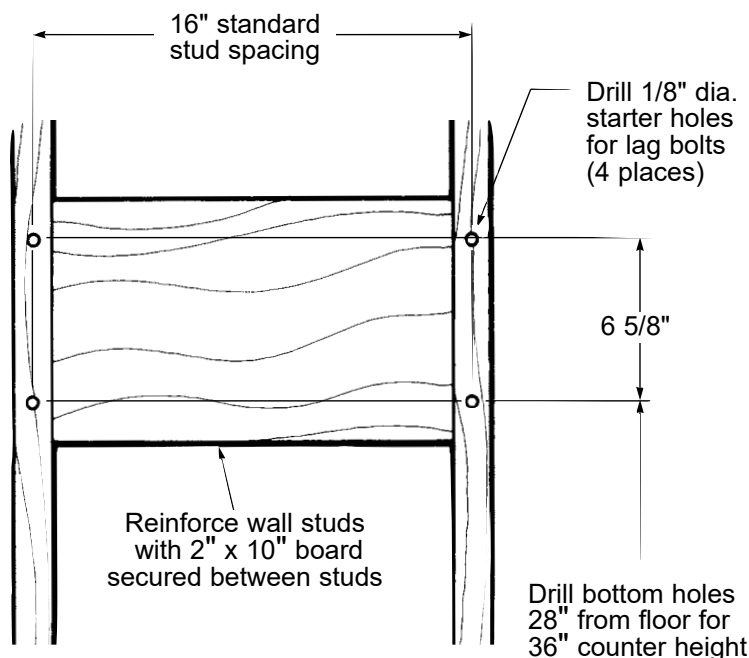


Pre-drill holes for the lag bolts as shown. To mount the table at standard 36" counter top height, drill the bottom holes at 28" from floor level.

When bolting the table to the wall, place large flat washers between the bolt head and table mounting plate.

If mounting the table to a masonry wall, appropriate anchor hardware should be used.


Note

Refer to chase installation manual for mounting hole locations in chase cabinet. To install table to chase, use hardware included with chase cabinet. (3/8" bolts, washers and nuts). Also use required bracket included in 002-10368-00 kit.

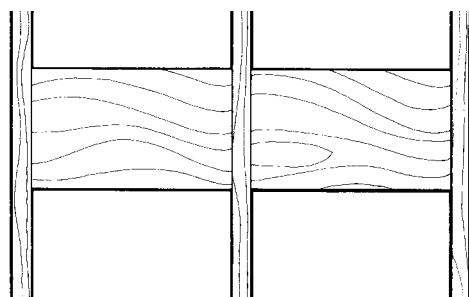

If the folding wall mount exam table will be mounted so that it does not correspond with the wall studs, secure 2" x 10" boards between wall studs as a mounting surface.


Secure table properly. The folding wall mount table is only as sturdy as its mounting.

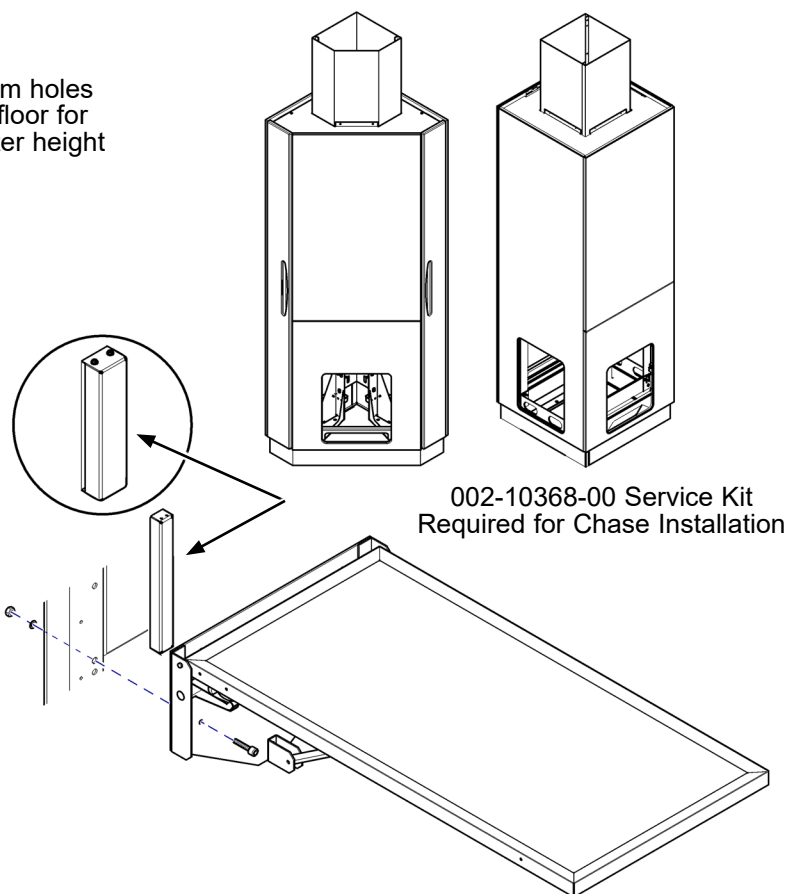

Leveling the Folding Wall Mount Exam Table




Drill and insert screw as shown to insure that the table remains level

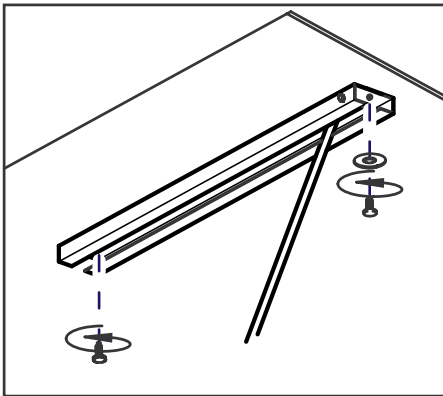


1.

If top is not level after mounting - Loosen screws at each end of the slide bracket.

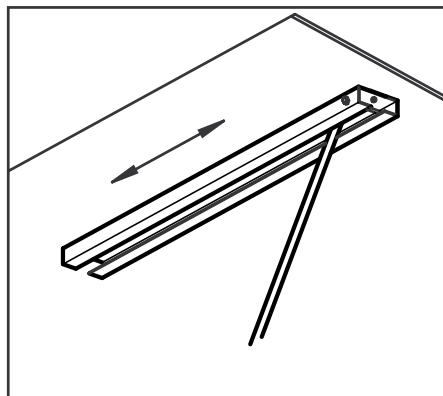


2.

Reposition the slide bracket until the table top is level, then tighten screws.

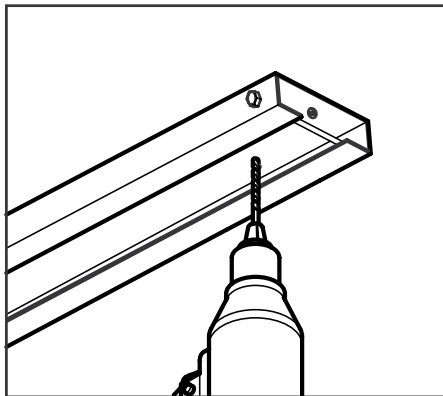

3.

Drill a pilot hole 1/8" diameter, 1" deep in the wood through the center hole of the bracket.

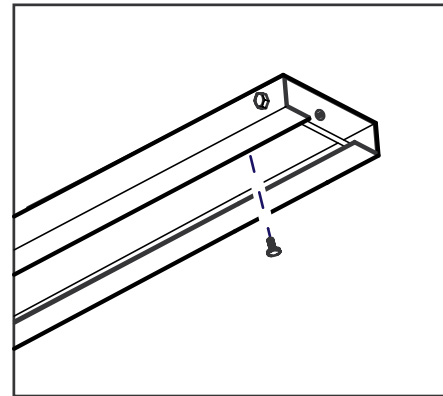

4.

Install the 1" screw (included) into the pilot hole to hold the bracket in place..

Midmark Corporation - Carthage Facilities

1000 Civil War Road
Area 5
Carthage, MO 64836
1-800-643-6275