



Model 110-3110-00
Digital Veterinary Scale
Calibration Procedure

Required: 50 lb. calibration weight

Begin with scale OFF

Press and Hold **ON/OFF** key while pressing **TARE** key three times to display SEt, display will then change to Unit.

Press **HOLD** key five times to display LbCA

Press **TARE** once to display OFF

Press **HOLD** once to change display to on

Press **TARE** once to enter calibration mode and display zero count.

Press **TARE** once more, display clears to show zero weight.

Place 50 lb. calibration weight on scale, displays span (not the weight)

Press **TARE**

Displays CAL

When calibration is complete, scale returns to normal weighing operation.