



# Peninsula Folding Lift Table Manual



ArtHalf

## ***Table of Contents***

Pre-Installation . . . . .	2
Parts Identification . . . . .	2
Installation . . . . .	3
Installation with Scale Option . . . . .	4
Scale Features/Operation . . . . .	4
Troubleshooting . . . . .	5
Care & Cleaning . . . . .	6
Specifications . . . . .	6
Product Warranty . . . . .	7
Product Satisfaction Policy . . . . .	7

# Pre-Installation



## WARNING

Do not move the Peninsula Folding Lift table by yourself. Improper handling may result in injury or damage to the product. Operator should never place foot on table base or under the table top.



## Equipment Alert


Weight limit on the table is 225 lbs..

## Note

Save all packaging until certain you are satisfied with the product.

Open packages carefully. Inspect the shipping carton and contents for any visible damage.

# Parts Identification

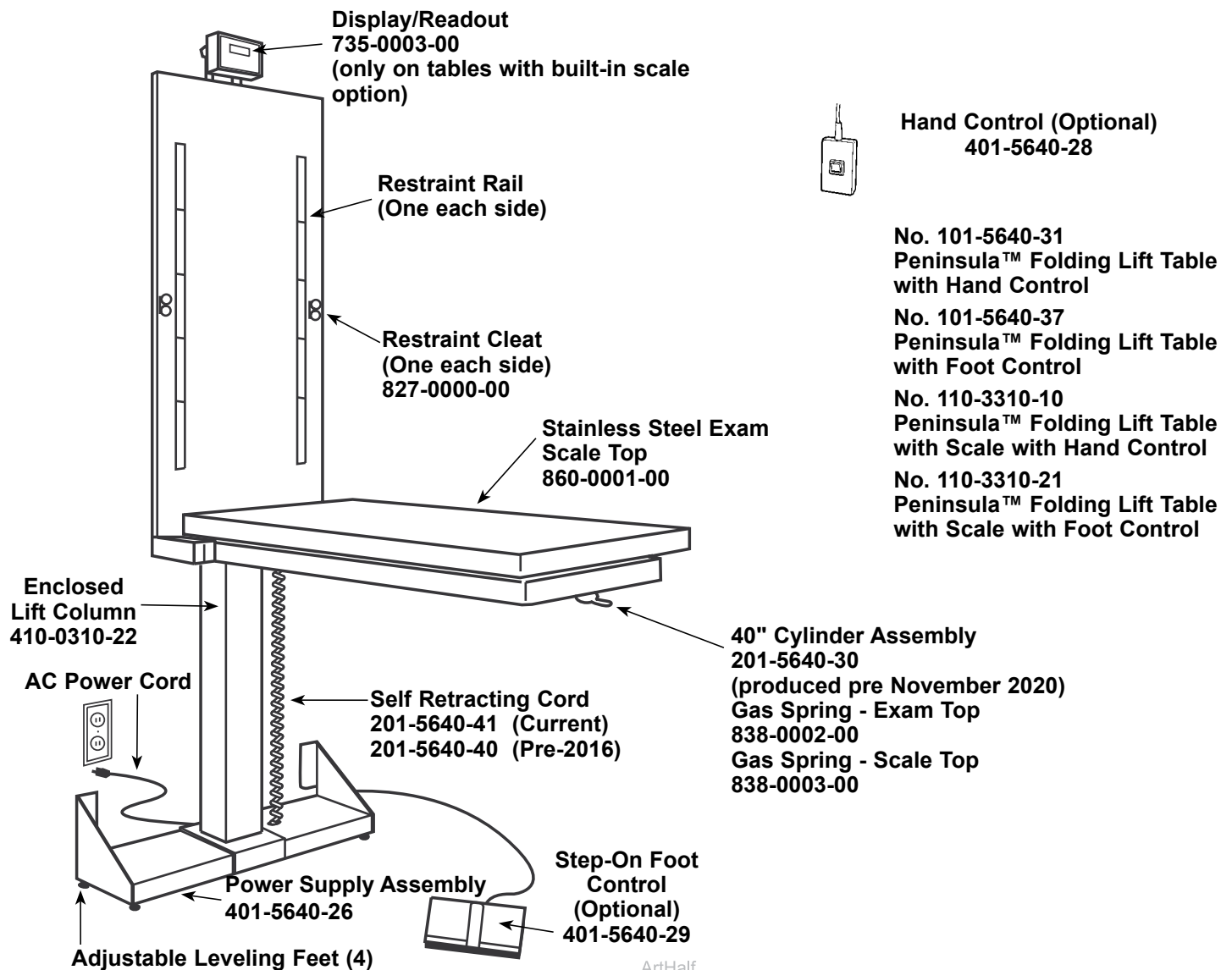

# Installation

Keep the table folded during installation. Lift Peninsula™ Folding Lift Table from base and column.

## Mounting the Peninsula™ Folding Lift Table:

Midmark recommends consulting your contractor before mounting table to the wall.

Hardware should meet minimum requirements: 3/8" diameter, 600 lb. shear capacity, 1,600 lb. pull capacity.



After installation, your table is ready for operation. Plug in the power cord at the base of the column. Remove the safety pin (shown at right below).

To lower the table firmly take hold of the exam surface and pull the tabletop down to the horizontal position, once all the way down gravity will hold the tabletop in the horizontal position.



To raise the exam top, pull up on the exam surface, when the tabletop reaches a 60 degree angle the gas spring dampening will activate and the top will slowly self-close from this point.

To raise or lower the table to desired height, depress the up or down switch of the hand or foot control.

If you purchased a scale top model, continue with the instructions on the next page. If you have an exam top only, go to page 6.

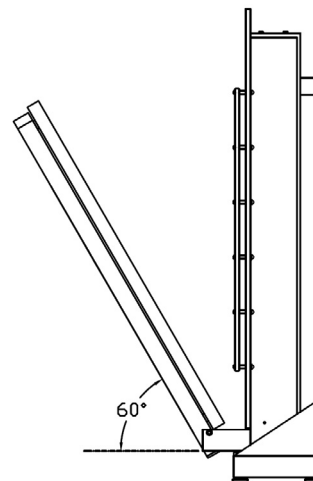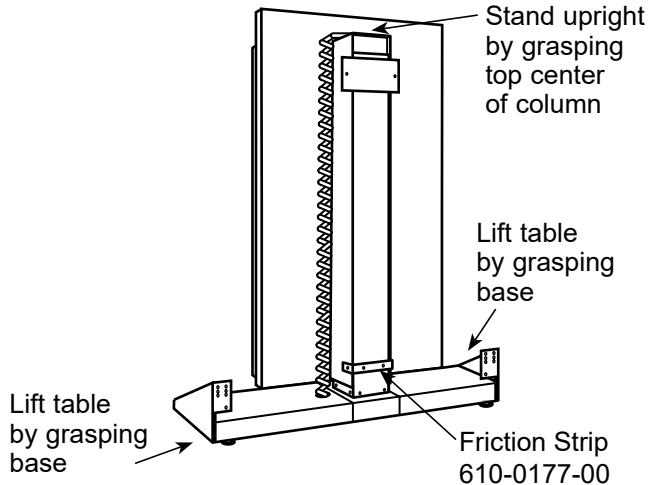

ArtHalf

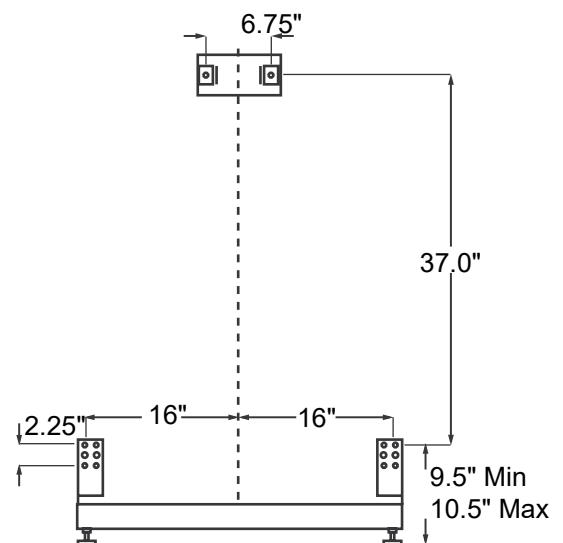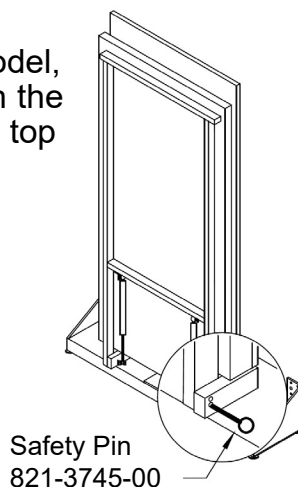


Mounting Template


ArtHalf

# Installation with Scale

Refer below for connecting instructions if installing a lift table with the optional scale built-in.

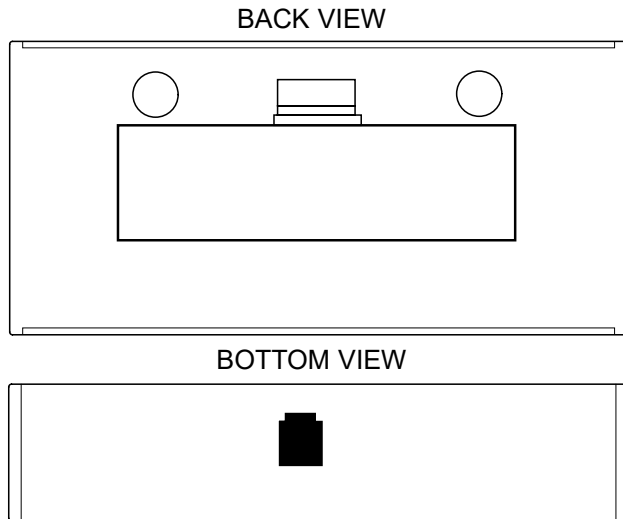

1. Remove the scale/readout display from the small box enclosed within the carton on the pallet.
2. Hang the scale readout/display over the top back of the stainless steel exam top.

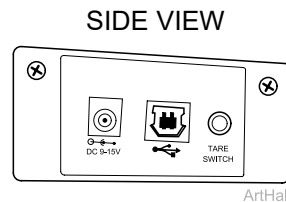

## Scale Features/Operation


### Key Functions

#### ON/OFF:

1. Press and release **ON/OFF** key to turn scale On or Off.

#### UNIT:

1. Press the **UNIT** key to change the weighing units to the alternate units of measurement.

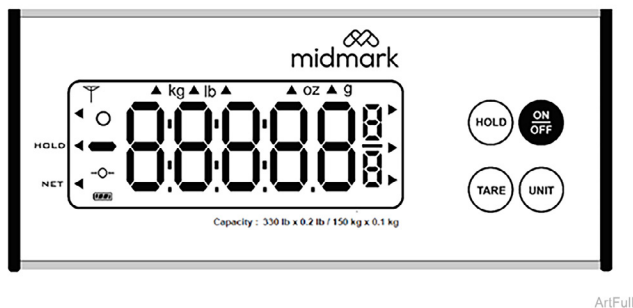

#### TARE:

1. Press the **TARE** key to zero the weight display or to tare the weight of a container (e.g. box or crate).
2. Press and hold the **TARE** key to enter the Configuration mode.

#### HOLD:

1. Press to turn the **HOLD** function On or Off.
2. Press to toggle between the settings for each configuration parameter, and then advance to the menu.



**DO NOT** operate the keypad with pointed objects (pencils, pens, etc).  
Damage to keypad resulting from this practice is **NOT** covered under warranty.

# Troubleshooting

---

## Display Annunciators:

The display annunciators are turned on to indicate that the scale display is in the mode corresponding to the annunciator label or that the status indicated by the label is active.

### **O**

This is turned on when the weight display is stable.



This is turned on when a negative (minus) weight is displayed.



This is turned on when the scale weight has been zeroed.

### **HOLD** ◀

This is turned on after pressing the **Hold** key.

### **NET** ◀

This is turned on after pressing TARE key to tare weight of item on scale.

### **kg**

This is turned on to indicate that the display weight is in kilograms.

### **lb**

This is turned on to indicate that the display weight is in pounds.

In the event that a problem occurs, a Midmark technician is ready to assist you. Call Midmark for assistance before attempting any repairs.

## In The Event Of A Problem:

In the event that a problem occurs, Midmark Technical Support is ready to assist you. In order to save time, please check the following items before calling:

- Does the readout display have a message that indicates what the problem is? Refer to “Indicators” at left for descriptions.
- Check all cables/connections.

**Midmark Technical Support: 1-800-643-6275**

# Care & Cleaning

---

The stainless steel components may be cleaned with any commercially available product recommended for use on stainless steel. Frequent cleaning will prolong the life and help maintain the natural luster of the stainless steel.

To prevent scratching, use a soft cloth to clean the scale. If scrubbing is necessary, use a nylon or plastic type scrubber. When cleaning, always rub in the direction of the “grain” of the metal.

To remove minor scratches, we recommend using a 3M ScotchBrite® scouring pad rubbed lightly in the direction of the “grain” of the metal.

**Never use steel wool or steel brushes** when cleaning your stainless steel components. Steel particles may become embedded into the stainless steel and cause rust.

**Avoid use of chlorine bleach for cleaning.** Chlorides, bromides, iodides and thiocyanates will cause pitting, corrosion and discoloration of the metal. If using bleach, clean thoroughly with water to completely remove bleach residue.

**Avoid excessive moisture.**  
**The Peninsula™** Folding Lift Table contains electronic components that may be damaged by excessive exposure to moisture. When cleaning or mopping the floor, do not leave the table in standing water.

# Specifications

---

## Dimensions

Exam Surface:	44" x 22"
Exam Height:	adjusts 8 1/2" to 40"
Base:	34" x 10 1/2"
Table Top Extended:	approx. 60"
Maximum Height:	90" when raised
Lift Capacity:	225 pounds
Scale Capacity (optional):	330 pounds
Scale Accuracy:	Within 0.2% at 300 pounds
Electrical:	Operates on standard 115 VAC

# ***Product Warranty***

---

This product is warranted for 1 year from the date of delivery to the original purchaser only. If equipped with the Midmark Scale, the scale is warranted for 1 year. Warranty is only applicable if customer complies with all instructions and specifications furnished by Midmark relating to installation, care, and application. Customer agrees that they will not modify, misapply, or misuse product in any manner which deviates from the Midmark instructions. Any repairs, alterations, or services provided by parties other than Midmark or its authorized representatives may void warranty. The buyer is responsible for all freight charges on returned items.

# ***Product Satisfaction Policy***

---

In the event that the customer is not fully satisfied with the products purchased, Midmark may, at its own discretion, arrange for the customer's account to be credited (excluding shipping charges) or replace the product. The customer must notify Midmark of any claim of nonconformity or defect in writing within 30 days from the date of receipt. The product must be returned to Midmark in its original packaging, undamaged and showing no unusual wear within 20 days of the customer's written notice.

To return an item, follow these simple instructions:

- Call Midmark for complete instructions (1-800-643-6275)
- A sales representative will give you a Return Authorization Number
- If a Restocking Fee applies, you will be informed at this time
- Label the boxes with this Return Authorization Number  
(All items must have a Return Authorization Number)
- Return the items, postage/freight prepaid

If necessary, the Midmark sales representative will assist you with freight arrangements.

Note: Custom orders and/or custom colors may not be returned.

**Midmark Corporation - Carthage Facilities**

1000 Civil War Road

Area 5

Carthage, MO 64836

[1-800-643-6275](tel:1-800-643-6275)

[www.midmark.com](http://www.midmark.com)

