

INSTALLATION INSTRUCTIONS

For Tray Option Kit #153218


PROCEDURE:

1. Loosen the set screw in the post.
2. Slide out the instrument holder assembly.
3. Assemble the parts as shown in the Illustration.



NOTE: Before sliding the assembly back into the post, check the movement of the tray arm. Loosen or tighten the socket head cap screw to adjust the tension.

4. Slide the assembly back into the post.
4. Tighten the set screw securely.

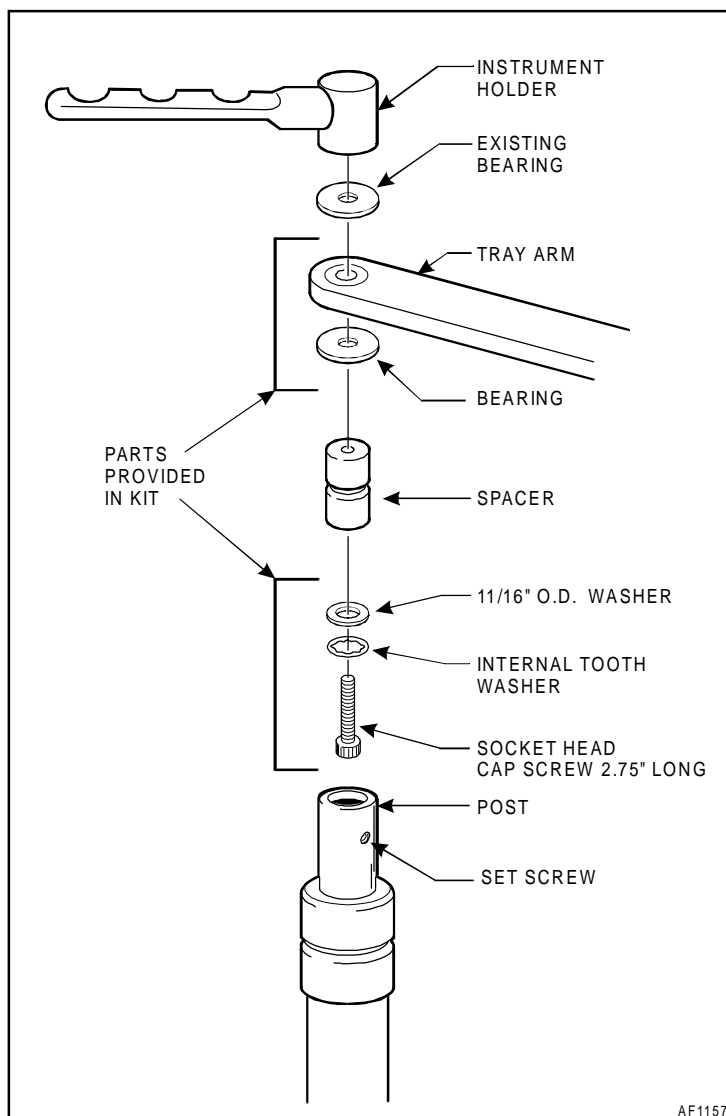

Knight®
by Midmark

Midmark Corporation • P O Box 286 • 60 Vista Drive
Versailles • Ohio • 45380-0286 • www.midmark.com
Phone: 937-526-3662 Fax: 937-526-5542

TPW1764 Rev. B