

Installation Instructions

for the *UNILITE CURING LIGHT*

J BOX CONNECTIONS

1. Inside the J box - Remove the plastic bag that is attached to the curing light power supply. This bag contains the connectors used in this procedure.
2. Connect the red wire coming from the umbilical to the red wire from the pressure switch (See Photo 1).
3. Connect the green tubing to the top of the pressure switch (See Photo 1).
4. Connect the black wire from the umbilical to the black wire from the power supply (See Photo 2).

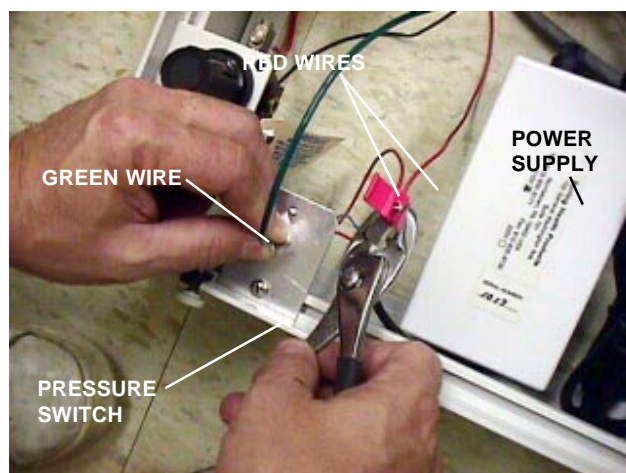


Photo 1

CHAIR MOUNT CONNECTIONS (LR UNITS ONLY):

1. Inside the J box - Remove the plastic bag that is attached to the curing light power supply. This bag contains the connectors used in this procedure.
2. Connect the red wire coming from the chair umbilical to the red wire from unit umbilical. Connect the black wire from the chair umbilical to the black wire from the unit umbilical (See Photo 3).
3. Connect the green tubing from the chair umbilical to the green tubing from the unit umbilical (See Photo 4).

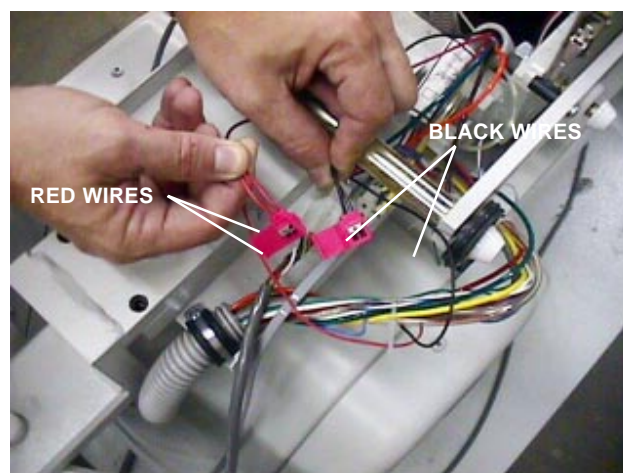


Photo 3

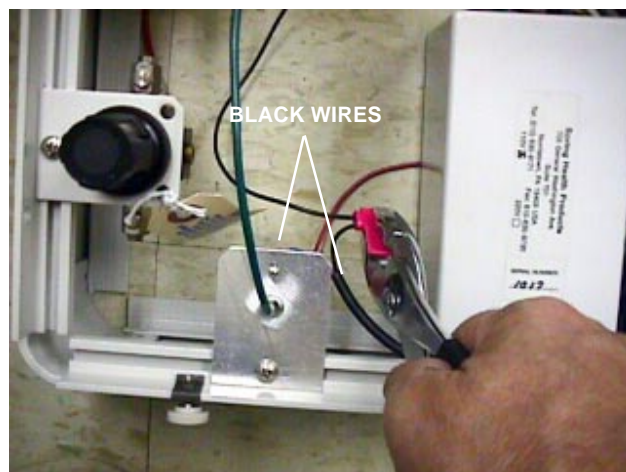


Photo 2

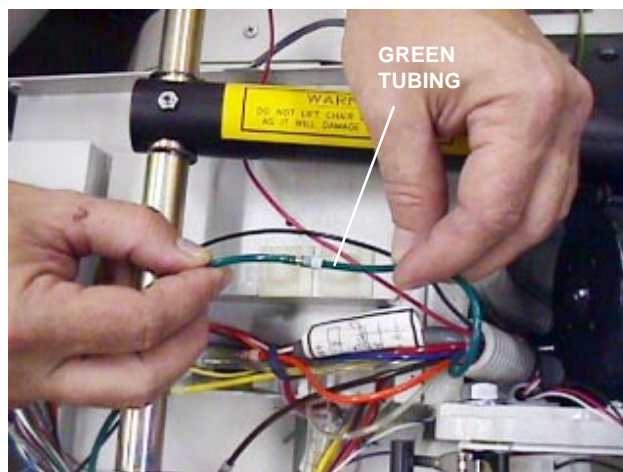


Photo 4

CONSOLE CONNECTIONS (CONSOLE UNITS ONLY):

1. Connect the red wire coming from the umbilical to the red wire from the unit (See Photo 5).
2. Connect the black wire coming from the umbilical to the black wire from the unit (See Photo 5).
3. Connect the green tubing from the umbilical to the green tubing from the unit, if this connection is not already made.

CURING LIGHT:

Assemble the curing light and position in the handpiece holder as shown in Photo 6.

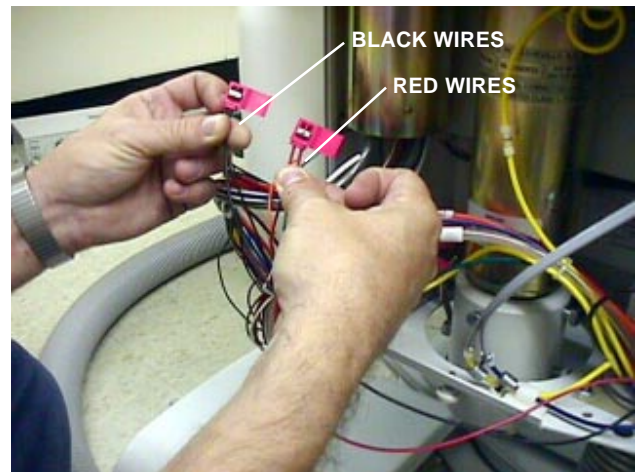


Photo 5



Photo 6

Knight®
by Midmark

Midmark Corporation • P O Box 286 • 60 Vista Drive
Versailles • Ohio • 45380-0286 • www.midmark.com
Phone: 937-526-3662 Fax: 937-526-5542

TPW1777 Rev. B